

AC440 Quick Key Guide

Quick Keys	Tone	Speech
1	Output CH1	Output CH1
2	Input CH1	Input CH1
3	Output CH2	Output CH2
4	Input CH2	Input CH2
5	Man Ch1	Man Ch1
6	Rev Ch1	Rev Ch1
7	Man Ch2	Man Ch2
8	Rev Ch2	Rev Ch2
9	Single Pulse	
0	Multi pulse	
Shift + S	Simultaneous	Simultaneous
Shift + A	Alternate	Alternate
R	Right	Right
L	Left	Left
S	Store Heard Thresh	Store
N	Not heard	No Response
O		O = Speech Reset
M	Mouse mode	
F1/Esc	Talk Forward On/Off	Talk Forward On/Off
F2	Speech Screen	
Shift + F2		Pure Tone Screen
F3	Extended Range	Extended Range
F4	HF	
F5	HF Zoom	
F6	Single/Dual Audio	
F7	MF	
F8	Synch Ch1/Ch2	Synch Ch1/Ch2
F9	Report	Report
F10	Edit	Edit
F11	TBD	
F12	Talk Back	Talk Back
Shift +1	Weber Test	Weber Test
Shift +2	MLD Test	MLD Test
Shift +3	MHA	MHA

AC440 Quick Key Guide

Shift +4	HLS	HLS
Shift +5	SISI	SISI
O	Output Drop CH1	Output Drop CH1
Alt+O	Output Drop CH2	Output Drop CH2
I	Input Drop CH1	Input Drop CH1
Alt+I	Input Drop CH2	Input Drop CH2